

**Product name/ Trade Name:**

Tubigrip<sup>®</sup> (excluding Size M)

Tubigrip<sup>®</sup> M Natural 10m (Size M only)

ELASTICATED TUBULAR SUPPORT BANDAGE

**Product description & Indications for Use:**

Elasticated Tubular Bandage BP Type A\*

- Easy to apply (0.5m/1m cartons only)
- Stays in place (0.5m/1m cartons only)

Fibre Content: Cotton, Elastodiene and Polyester

Provides firm support for sprains, strains and swelling. (0.5m/1m cartons only)

BP Type A \* English only, not to be included in other languages or on the Size M label

**Contraindication(s): – N/A**

**Warning(s):**

**Caution: This product contains natural rubber latex which may cause allergic reactions, including anaphylactic responses.**

**Precaution(s):**

Do not use Tubigrip if known hypersensitivity to the fibre content. If your condition worsens stop use and consult a health care professional. (0.5m/1m cartons only)

Do not use Tubigrip if patient/user has known hypersensitivity to the fibre content. If your condition worsens stop use and consult a health care professional. (10m only)

Keep clean, if soiled discard and replace bandage.

**Instructions for use:**

Consult size guide. Cut the bandage to twice the length of the affected limb, if required make 2 slits for a thumb hole. Pull over the limb, then double back. (Excluding Size M)

Disposal should be handled according to local environmental procedures.

**Special storage conditions and handling conditions**

Store below 25°C (use symbol on ARS)

**Other information**

If any serious incident has occurred in relation to the use of Tubigrip, it should be reported to Mölnlycke Health Care.