ARION Easy-Slide ARM

EN	Donning aid for medical compression arm sleeves	Instructions for use	2
DE	Anziehhilfe für medizinische Kompressionsarmstrümpfe	Gebrauchsanleitung	4
NL	Aantrekhelp voor medische compressiearmkousen	Gebruiksaanwijzing	6
FR	Aide à l'enfilage de manchons de compression médicale	Mode d'emploi	8
ES	Colocador de mangas de compresión médica	Instrucciones de uso	10
PT	Auxiliar para vestir mangas de compressão médica	Instruções de utilização	12
IT	Dispositivo per mettere le maniche compressive medicali	Istruzioni per l'uso	14
DA	Hjælp til påtagning af kompressionsærmer	Brugsanvisning	16
SV	Påklädningshjälpmedel för medicinska kompressionsärmar	Bruksanvisning	18
NO	Påføringshjelp for medisinske kompresjonsærmer	Bruksanvisning	20
FI	Pukemisen apuväline lääkinällisille kompressiohihoille	Käyttöohjeet	22

Arion Easy-Slide Arm

Donning aid for medical compression arm sleeves

Important information before you start using the product:

- A.** Useful tips:
- Wear special non-slip gloves when using the product. This gives you maximum grip on the stocking and reduces the physical strength needed.
 - In case the compression garment has a silicon border, make sure to turn it outwards.
- B.** Make sure the product is ready for use:
- Fold back the extra fabric into the „green bag“
 - Align the fold line with the top of the orange labels

Donning instructions:


1. Slide the Easy-Slide Arm over your arm.
2. Slide the compression arm sleeve over the Easy-Slide Arm.
3. Slide the arm sleeve into the correct position and place the upper border correctly on your skin.
4. Hold the arm sleeve firmly at the elbow with your hand and hang the orange loop on a door handle. Slowly pull the Easy-Slide Arm out of the arm sleeve by gently backing up.

SIZING

REF	Size	Arm length*	Max. product circumference**
Eo 1505	Medium	59 - 67 cm / 23" - 26.5"	45.6 cm / 18"
Eo 1506	Large	> 67 cm / > 26.5"	57.6 cm / 22.5"

* Arm length from the tip of the middle finger to the armpit.

** For a full-length distribution of the Easy-Slide Arm over your arm, we recommend that the circumference of the widest part of your arm is min. 1 cm/0.4" smaller than the max. product circumference. If this is not possible with the current sizes available, please ask your reseller or Arion for a made-to-measure product.

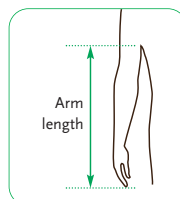Easy-Slide Arm from Arion Slide Solutions minimizes friction between compression garment and skin thanks to the product's unique sliding properties provided by the special material it is made of and by its coating.

EASY, SAFE AND SIMPLE SOLUTION

- Reduces physical strain for both compression wearers and caregivers
- May contribute to a longer life for the garment and to maintaining the original compression longer by preventing damage from fingers or nails and avoiding overstretching and distortion
- Promotes independence of compression wearers and supports therapy compliance

COMPOSITION

- Product: Polyamide 93%, Polyurethane 3%, Silicone 2%, Polyester 2%
- Package: Chromo carton GC1 (PAP21) 

CONTRAINDICATIONS

- Do not use the product if you are allergic to one of its component (see composition).
- Open wounds should be covered with an appropriate dressing before using the product.

WARNINGS

- **Suffocation hazard:** Keep the product away from babies, children, and pets.
- **Slipping hazard:** Do not step on the product. Highly slippery material.
- **For external use only /** Intended for skin contact. Seek medical attention if irritation develops.

IMPORTANT

-  Please **read the instructions for use carefully**. If you have any questions, please contact your reseller.
- Please **keep the package and the instructions for use during the time of use of the product**.
- All **serious incidents** that occur when using the product **should be reported** to the manufacturer and the competent supervisory authority.
- When using the product it is important to **use the right product size** for the user. Using an incorrect size will prevent the product from working properly, and it may become damaged or wear out too quickly.
- **Use the product on a single person**. If the product needs to be used on more than one person, disinfect the product before with a 70% ethanol solution.

STORAGE

Keep away from direct sunlight or other heat sources. 

CARE INSTRUCTIONS

- Wash by hand, at max. 30°C/86°F.
- Do not bleach.
- Do not tumble dry. Dry the product only with a towel.
- Do not iron.
- Do not dry clean.
- Disinfection possible with a 70% ethanol solution.

GUARANTEE

This product is guaranteed for a period of six months from the date of purchase against defects resulting from faulty material or construction. Within the guarantee period, the product will be exchanged free of charge in the case of manufacturing and/or material defects. Any exchange under the guarantee will only be carried out if it can be demonstrated by means of a purchase receipt that the date on which the complaint was lodged falls within the guarantee period. The guarantee does not apply if the defect is due to improper use of the product or use on several patients. To use this service you need to contact your supplier. You are asked to return the product washed and to follow the care instructions.

RECOMMENDATION FOR USE

 The manufacturer recommends replacing the product six months after the first use if the product is used regularly.

You can write the date of first use on the back of this document.

Watch our **instruction video** here: uqr.to/ESarmEN
or scan this QR code:

More information on www.slide-solutions.com

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Arion Slide Solutions B.V.