To watch video instructions, please visit our website: www.sigvaris.com/videos

- DND:** Net weight: 700 gram / Gross weight: 840 gram / Box size (mm) LxWxH: 125 x 125 x 160 mm
- CONE:** Net weight: 470 gram / Gross weight: 720 gram / Box size (mm) LxWxH: 590 x 115 x 115 mm

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If you look for further distributors please consult: www.sigvaris.com

SIGVARIS and SIGVARIS GROUP are a registered trademarks of SIGVARIS AG, CH-9014 St. Gallen/Switzerland, in many countries worldwide.
© 2020 Copyright by SIGVARIS AG, CH-9014 St. Gallen/Switzerland

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DOFF N' DONNER SET, consisting of:
A: DND
B: CONE
Intended purpose: Facilitates donning and doffing of medical compression stockings
Contraindications: No known contraindications

Suitable for the following models:
A-D Calf stocking, open / closed toe, soft top or grip top
A-G thigh stocking, open / closed toe, soft top or grip top

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EN: Fill the sterilization bag with a disinfectant (DND should be lightly soaked in the liquid).

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EN: Move the sterilization bag up and down so that the disinfectant is distributed well and evenly through the DND.

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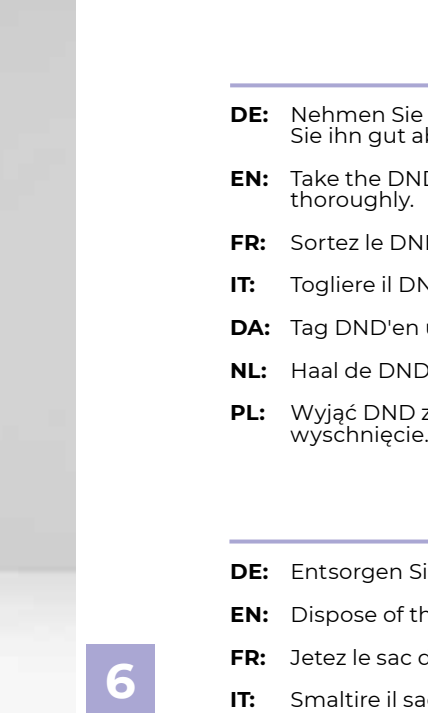
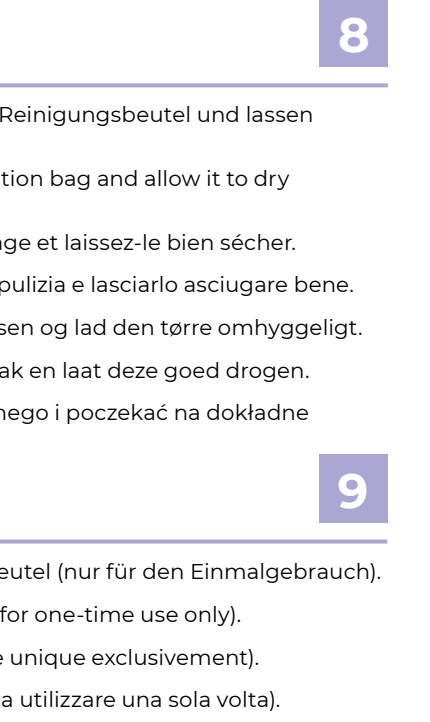
EN: Take the DND out of the sterilization bag and allow it to dry thoroughly.

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EN: Dispose of the sterilization bag (for one-time use only).

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EN: All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

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EN: The DOFF N' DONNER (DND) is a very soft device that allows you to quickly and painlessly convey medical compression stockings over an arm or a leg. The DND is suitable both for self-application and assisted application. The CONE simplifies the rolling of a compression stocking onto the DND. With the aid of the CONE, the medical compression stocking can also be easily unrolled from the DND. By virtue of the integrated suction cup, the CONE can be attached well to smooth surfaces.

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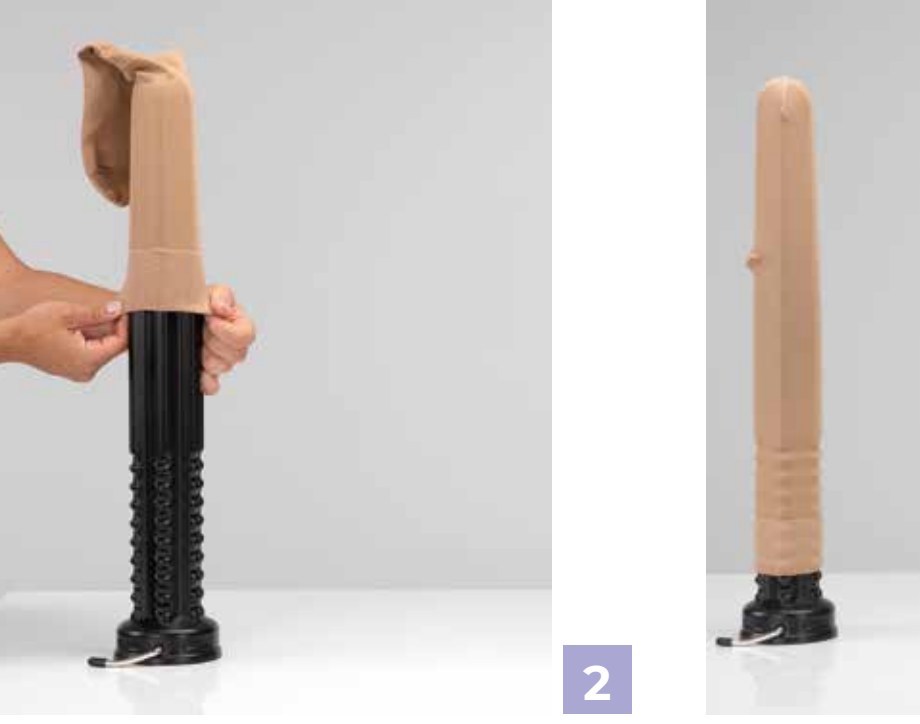
EN: Clamp the CONE to a smooth surface (press the lever down, CONE adheres firmly).

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EN: Place the compression stocking with the finished top edge (soft top or grip top) over the CONE. The inside of the stocking rests on the CONE.

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EN: Place the DND over the CONE and roll it downward. Pull the finished edge of the compression stocking up over the DND.

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EN: Thigh stocking: Pull the finished edge of the compression stocking beyond the tip of the CONE, so that the stocking is wrinkle-free. Grasp the DND and pull it upward until the compression stocking is rolled completely onto the DND.

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EN: Stretch the toe seam, so that it is clearly visible from the outside. Hold the DND in such a way that the heel of the compression stocking is visible from the outside and is facing the floor.

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EN: Place the toe seam on the toes. The heel of the rolled-up compression stocking points downward.

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